Fibrinogen VET test kit

For veterinary use only!

Veterinary test kit for quantitative in vitro determination of fibrinogen in citrated plasma or lithium heparin plasma with a solo analyser.

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Order number: VT 0120
Order number: VT 0121
Order number: VT 1200

Indication

Fibrinogen VET test kit
Fibrinogen VET test kit
Fibrinogen VET control kit

Kit size

16 tests/kit
6 tests/kit
1 x 1 ml

Test kit preparation: allow single test at least 10 minutes to warm up to room temperature (20 - 25 °C) by placing the test into the test kit rack. Return the test kit package back to the refrigerator.

Summary

The Fibrinogen VET test is intended for the quantitative in vitro determination of fibrinogen by immunoturbidimetric method for veterinary use. The fibrinogen antibody interacts with the fibrinogen in the plasma, forming immune complexes. The immune complexes cause an increase in light scattering which correlates with the concentration of plasma fibrinogen.

Method

Immunoturbidimetric assay measured at 340 nm wavelength.

Measurement Range

0 - 8 g/l (0 - 800 mg/dl) (Lot dependent)

Samples with concentrations higher than the upper limit of the measurement range must be diluted 1 + 2 with physiological saline (0.9% NaCl solution), e.g. 20 µl sample + 40 µl 0.9% NaCl solution and the result then multiplied by 3.

Sample Material

5 µl citrated plasma or lithium heparin plasma is required for this assay. Centrifuge samples to remove plasma from cells immediately after collection!

Test Kit

ERS cuvette filled with buffer reagent

ERS cap filled with antiserum reagent

Stability and Storage

Stable until the expiration date stated on the label when stored in unopened vacuum package at 2 - 8 °C.

Opening the vacuum package may limit the reagent stability to three months (stored at 2 - 8 °C) from the date of opening.

DO NOT FREEZE!

Warnings and Precautions

DO NOT INGEST! Avoid contact with skin and eyes. Contains sodium azide, which may react with lead or copper plumbing to form explosive compounds. Observe all necessary precautions for the use of laboratory reagents.

Waste Management

Always refer to local legal requirements.

Reference Range

Horse: 1.5 - 3.3 g/l (150 - 330 mg/dl)

It is recommended that each laboratory establishes its own reference range.

Quality Control

For internal quality control the Eurolyser Fibrinogen VET control kit is recommended. Order number: VT 1200

Precision

Reproducibility within-run:

Control; N = 20; mean = 338 mg/dl; CV = 2.96%;

Correlation

Horse sample correlation:

y (solo FIB) = 1.0369 x (COBAS DAKO FIB) - 11.239; R² = 0.9683;

Interferences

The test system has been analysed for various interferences. Criterion was the recovery within 10% of initial values.

Bilirubin	30 mg/dl
EDTA	1000 mg/dl
Haemoglobin	1000 mg/dl
Sodium citrate	1000 mg/dl
Triglyceride	2500 mg/dl

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English

ATTENTION!

Allow single test at least 10 minutes to warm up to room temperature (20 - 25 °C)!

1. Preparing the test system

- 1.1 Insert RFID card
- 1.2 Place ERS cuvette into test kit rack
- 1.3 Place ERS cap into test kit rack
- 1.4 Press "Measurement" button and enter all required information using the touchscreen

2. Sample preparation

- 2.1 Choose sample type: Ensure that "*Pipette5µ*" is selected when performing a sample measurement. When performing a control measurement please make sure "*CtrlPlasma*" is selected
- 2.2 Aspirate 5 µl sample material from centrifuged sample tube

3. Test processing

- 3.1 Dispense 5 µl sample INTO THE LIQUID in the ERS cuvette
- 3.2 Firmly apply ERS cap on ERS cuvette
- 3.3 Place ERS cartridge into the solo analyser
- 3.4 Start automatic sample processing by pressing the  start button on the solo analyser