





Installing OSi Software onto i-STAT Alinity v Using a USB Drive

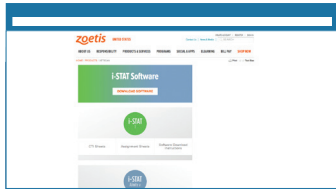

Go to:
<https://www.globalpointofcare.abbott/us/en/lp/apoc/vetscan-istat-system-software-update.html//>

> Select Download Software

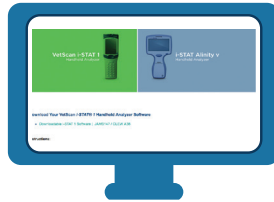

Navigate to Download Your i-STAT Alinity v Handheld Analyzer Software
> Select Downloadable i-STAT Alinity v OSiXX System Software




Save software to a USB drive

* Requires FAT32 formatted USB 2.0 memory stick
* File must be saved on the top-level directory of the USB drive




Place instrument into the Base Station and power on the instrument

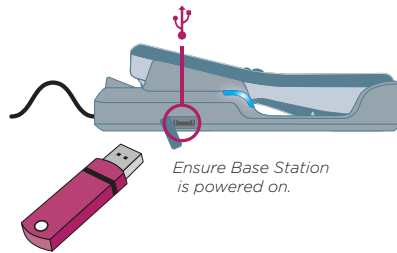


Ensure Base Station is powered on.

Insert USB drive into the USB port in the Base Station

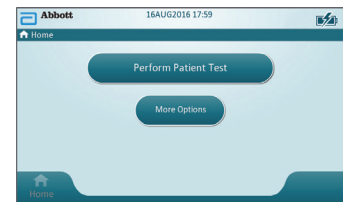


From the Home Screen:

- > More Options
- > Instrument Options
- > Software Installation
- > Install from USB

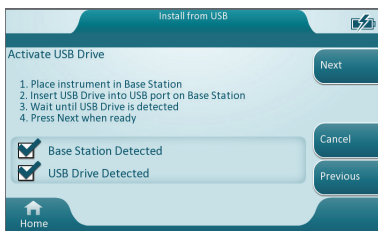


Follow prompts until transfer is complete

SUCCESS

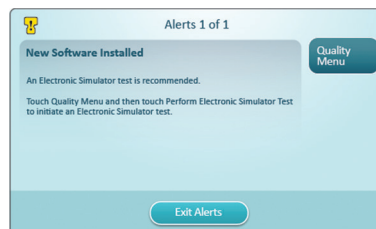

OSi Software Installation Results
SUCCESS

FAILURE

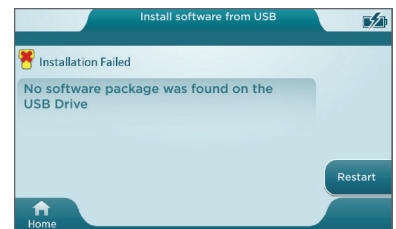

OSi Software Installation Results
FAILURE

If additional assistance is required, please contact Zoetis Technical Support at 1-800-822-2947. Support is available 24 hours/day, 7 days/week.